CONFIRME®

Teste Plus

TESTE DE GRAVIDEZ PRUEBA DE EMBARAZO PREGNANCY TEST

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Sensitivity: 25 mIU/ml
Sample: Urine
Presentation: 1 test

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PURPOSE

The Confirme® Teste Plus is a safe, easy and fast test for the detection of hCG in urine, any time of day, from the first day of missed period, but you can perform it up to 4 days before according to the follow study results (see the results obtained in previous studies in item questions and answers).

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PRINCIPLE

The Confirme® Teste Plus is a qualitative immunoassay employing the sandwich method (5-6) to detect the human chorionic gonadotropin hormone (hCG) in urine. The membrane is impregnated with monoclonal anti-hCG antibodies in test area and with polyclonal antibody anti-hCG in the control area. During the test, if urine has the presence of hCG, it reacts with the anti-hCG monoclonal antibody colored conjugate pre-fixed to the membrane. The mixture goes through the membrane by capillary action to react with the anti-hCG fixed and forms a pink line at the test region. The presence of this pink line indicates a positive result, while its absence indicates a negative result. Regardless of the presence of hCG, the mixture or only the conjugate, continues to rise for the control area, reacting with the antibody in the control area, forming a pink line. The presence of this pink line in the control area serves to demonstrate that the volume and flow of the sample was adequate and appropriate, so as the functionality of the reagents.

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WARNINGS AND PRECAUTIONS

- The package containing the device should be opened only at the time of use.
- The package contains the device and a bag of desiccant that should be discarded.
- The test should be kept at a temperature of 2° C - 30° C.
- It is a self-test for guidance without diagnostic purposes.
- For "in vitro" diagnostic use. Not intended for internal use.
- Follow carefully the procedure for testing.

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- Cronometrador.

MATERIAL REQUIRED, BUT NOT INCLUDED

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CONTENT OF THE CONFIRME® TESTE PLUS FOR 1 TEST:

- 1 aluminum envelope containing a device (MIDSTREAM T HCG) and 1 desiccant bag (do not use, serves only for storage of the test)
- 1 Instruction for Use

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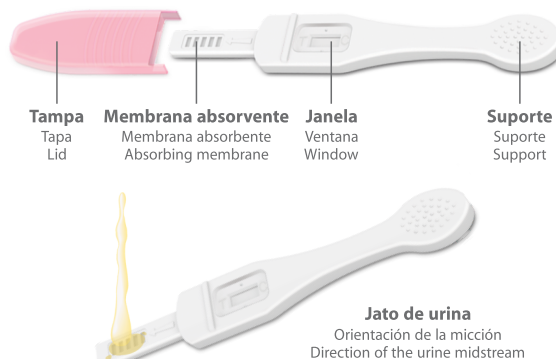
TESTING

1. Open the envelope and remove the test holding the support test.
2. Remove the lid.
3. Point the absorbent membrane towards your urine stream. Maintain this position for 10 seconds or until the urine penetrates the membrane absorbent. **Take care that the stream of urine does not point to the window. The urine can not be absorbed by the window. If this occurs the test can be invalidated.**
4. Put the cover back and rest the test on a flat surface. The lines may appear in 1 minute. Wait 3 minutes to confirm the result.

Or if you prefer, collect urine in a clean and dry recipient, dip the absorbent tip until the urine is absorbed. Remove from the urine and the lines may appear in 1 minute. Wait 3 minutes to confirm the result.

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INTERPRETATION OF THE TEST

POSITIVE: If after 3 minutes, 2 pink lines are seen (1 intense pink line and 1 strong or weak intensity line of the pink color), consult your doctor.

NEGATIVE: If after 3 minutes, just 1 pink line is seen.

INVALID: If no pink line is seen, the test should be considered invalid. The control line shows that the test was performed correctly.

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LIMITATIONS OF THE PROCEDURE

- Samples of very dilute urine, with low specific gravity, may not have representative levels of hCG. If pregnancy is suspected, collect the first morning urine and 48-72 hours later and analyzed.
- Very low levels of hCG (less than 50 mIU/ml) are present after fertilization. Because of significant numbers of miscarriages during the first trimester of pregnancy, a weak positive test result should be read in conjunction with other clinical and laboratory data.
- In addition to pregnancy, trophoblastic disease and other non-trophoblastic neoplasms including testicular tumors, prostate cancer, breast cancer and lung cancer have elevated levels of hCG. Therefore, the presence of hCG in urine determined by Confirme® Teste Plus must be used to diagnose pregnancy provided that the conditions mentioned have been discarded.
- Like all diagnostic tests, confirmation of pregnancy should only be confirmed by a doctor after being evaluated all clinical and laboratory reports.

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QUESTIONS AND ANSWERS

How does Confirme® Teste Plus works?

Confirme® Teste Plus detects the hCG (Human Chorionic Gonadotropin) hormone, which is present in the urine of pregnant women in larger quantities.

Confirme® Teste Plus is accurate?

Confirme® Teste Plus has an accuracy of 99.9%.

I have to use first morning urine?

No, you can use Confirme® Teste Plus at any time of day, but the first morning urine may have a higher concentration of hCG and therefore be more likely to give a positive result.

When should I get tested?

You can perform the Confirme® Teste Plus on the first day you miss you menstrual period, but you can perform it up to 4 days before according to the follow study results:

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If you perform the test on the above conditions and the result is negative, it is suggested that the pregnancy hormone is not detectable and therefore should repeat the test with at least 1 day of missed period.

Some medication can cause false results?

All drugs containing hCG (Human Chorionic Gonadotropin) can give false result.

If the result is positive?

If the result is positive, see your doctor.

If the result is negative?

If the result is negative, it means that the hormone hCG was not detected. If your menstrual period does not come within a week, repeat the test and if negative again, and you do not menstruate, see your doctor.

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QUALITY ASSURANCE

The Alamar Tecno Científica Ltda, obeying the Consumer Protection Code and for best product performance, recommends:

- The user must read and strictly follow the technical procedure;
- The materials be stored under indicated conditions;
- The accessories be in accordance with the request.

Before being released for sale, each batch is tested and approved, a sample being retained for future reference and quality control. Therefore if there is need for some information about the lot in question, the quality control is available. And any problems that may occur due to failure by the company, will be resolved without charge to the customer.

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1) Batzer, FR Fertility, Vol 34, 1, 1980. 2) Catt, Kjet al. J.L., J. Endocrinol. Metab., Vol 40, 537, 1975. 3) Braunstein et al., Vol 126, 678, 1976. 4) Lenton, EA, et al, R. Fertility and Sterility, Vol 37, 773, 1982. 5) Wide, L., Acta Endocrinol, 41 (Suppl.70) 75, 1962. 6) Dawood, MY, et al. Ob Gyn. Vol 126, 678, 1976. 7) Braunstein, GD, et al: Ann. Inter. Med Vol 78, pp. 39-45, 1973. 8) Engavil, E. Method in Enzymology, Vol 70, pp. 419-439, 1980. 9) Uotila, M., et al, Immunol. Methods, Vol 42, 11, 1981. 10) Data on file, Cal-Test Diagnostics, Inc. Chino, CA 91710, USA. 11) Meyer, HM, Jr., "Statement to be include in Directions for Use of Anti-HCG Serum" Memorandum, DEW, BOB, november 2, 1973.